

Vetario Intensive Care Units - Care and Cleaning



To get the best out of your Vetario we suggest you read the following guide to care & cleaning and display it near to your intensive care unit.

- Disconnect the intensive care unit from the mains power supply and oxygen supply where applicable during cleaning. Risk of electric shock! Ensure that all electrical parts are kept dry.
- Remove all debris from the floor.
- Wipe all internal surfaces with a soft cloth soaked in 100:1 Brinsea Incubation Disinfectant solution. If using an alternative cleaning solution it is best to avoid alcohol-based solutions.
- Make sure that all disinfectant is thoroughly rinsed off plastic parts with clean water. Disinfectant solutions may cause cracking of the plastic if not correctly rinsed off.
- Filters in the S40, S50, T40M & T50M should be inspected on a weekly basis and be cleaned if necessary. Remove all three filters and gently hand wash in warm water then allow them to dry before use. Filters need replacing approximately every six months. Evaporating blocks should be replaced regularly.
- The exterior of the Vetario intensive care unit may be cleaned with a damp cloth.
- Always clean the Vetario ICU before storage and ensure that the unit is totally dry inside and out or damage may occur to the components. It is advisable to run the Vetario for 24 hours before storage to ensure components are fully dried out.
- For more in-depth cleaning the heater enclosure may be safely removed to allow the heater and fan to be dusted - please refer to the product instructions for detailed information on how to do this. If you have lost your instructions they may be downloaded free of charge from the Vetario website.
- For deeper cleaning the base of the cabinet and the door may be removed by releasing the fasteners. The outside of the cabinet and the door may be cleaned with mild detergent and then disinfected with water based disinfectant solution such as Brinsea Incubation Disinfectant. Rinse and dry thoroughly.
- Replacement filters, evaporating blocks (S40, S50, T40M & T50M) and disinfectant solution are all available from www.vetario.co.uk

32-33 Buckingham Road, Weston Industrial Estate, Weston-super-Mare, North Somerset, BS24 9BG Tel: 0345 226 0120 or +44 (0) 1934 417523