



## Veterario Intensive Care Units - Care and Cleaning



**To get the best out of your Veterario we suggest you read the following guide to care & cleaning and display it near to your intensive care unit.**

- Disconnect the intensive care unit from the mains power supply and oxygen supply where applicable during cleaning. Risk of electric shock! Ensure that all electrical parts are kept dry.
- Remove all debris from the floor.
- Wipe all internal surfaces with a soft cloth soaked in a weak solution of water-based disinfectant (diluted in accordance with the manufacturer's recommendations) in the water reservoir. If using an alternative cleaning solution, it is best to avoid alcohol-based solutions.
- The exterior of the Veterario intensive care unit may be cleaned with a damp cloth.
- Make sure that all disinfectant is thoroughly rinsed off plastic parts with clean water. Disinfectant solutions may cause cracking of the plastic if not correctly rinsed off.
- Filters in the S40, S50, T40M & T50M should be inspected on a weekly basis and be cleaned if necessary. Remove all three filters and gently hand wash in warm water then allow them to dry before use. Filters need replacing approximately every six months. Evaporating blocks should be replaced regularly.
- Always clean the Veterario ICU before storage and ensure that the unit is totally dry inside and out or damage may occur to the components. It is advisable to run the Veterario for 24 hours before storage to ensure components are fully dried out.
- For more in-depth cleaning the heater enclosure may be safely removed to allow the heater and fan to be dusted - please refer to the product instructions for detailed information on how to do this. If you have lost your instructions, they may be downloaded free of charge from the Veterario website.
- For deeper cleaning the base of the cabinet and the door may be removed by releasing the fasteners. The outside of the cabinet and the door may be cleaned with mild detergent and then disinfected with water based disinfectant solution. Rinse and dry thoroughly.
- Replacement filters, evaporating blocks (for all 40 and 50 size models) and disinfectant solution are available from your Veterario distributor.